
# MRI Needle Guide SNG

de	-	Gebrauchsanweisung	.....	4
en	-	Instructions for use	.....	6
fr	-	Mode d'emploi	.....	7
it	-	Istruzioni per l'uso	.....	9
es	-	Instrucciones de uso	.....	11
pt	-	Instruções de utilização	.....	12
nl	-	Gebruiksaanwijzing	.....	14
da	-	Brugsanvisning	.....	16
no	-	Bruksanvisning	.....	17
sv	-	Bruksanvisning	.....	19
fi	-	Käyttöohje	.....	21
et	-	Kasutuskorraldus	.....	22
lv	-	Lietošanas pamācība	.....	24
lt	-	Naudojimo instrukcija	.....	26
pl	-	Instrukcja użytkowania	.....	27
cs	-	Návod na použití	.....	29
sk	-	Návod na použitie	.....	31
sr	-	Uputstvo za upotrebu	.....	32
ro	-	Instrucțiune de utilizare	.....	34
tr	-	Kullanım kılavuzu	.....	36

BIP Biomedizinische Instrumente und Produkte GmbH  
Am Brand 1  
82299 Türkenfeld  
Germany  
Tel.: +49(0)8193-93180  
Fax: +49(0)8193-6548  
[info@bipmedical.com](mailto:info@bipmedical.com)  
[www.bipmedical.com](http://www.bipmedical.com)

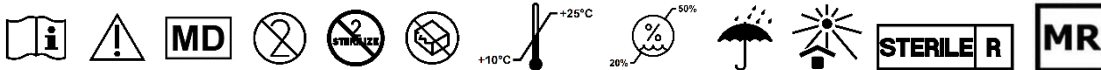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## ENGLISH

### Intended purpose

The MRI needle guide is intended to be positioned under MR control for guiding needles for rectal prostate biopsies. It is intended to be used only with a compatible Remote Controlled Manipulator (RCM). The MRI needle guide is intended for single-use and for transient application in body orifice.

#### • Indications

Indications are defined by the compatible RCM according to its application, such as anomalies in prostate tissue.

#### • Contraindications

Contraindications are defined by the compatible RCM according to its application and are not caused by the MRI needle guide itself.

#### • Residual risks and side effects

Residual risks and undesirable side effects are defined by the compatible RCM according to its application. General complications are bleeding, infection, injury to surrounding tissue and pain.

#### • Intended users

Intended users are health care professionals only (physicians). The user must be familiar with the use of the RCM.

#### • Intended patients

Intended patients are adult males.

### Clinical benefits

Clinical benefits are defined by the compatible RCM.

### Description

The MRI Needle Guide SNG is a disposable tube to guide a needle through the rectum to the prostate, with a handle to be connected to Soteria Remote Controlled Manipulator RCM (see Fig. 2). The tube itself is made of plastic, is not pre-calibrated and has no measuring function. The MRI Needle Guide SNG is MR compatible and detectable in conventional 1.5T and 3T MR imaging systems by its transparent, double-walled cylinder that must be filled with sterile water prior to application.

The MRI Needle Guide SNG is sterile, single-packaged in peel pouches and is available in specific diameter and length (see Tab. 1). Sales units are additionally packaged in protective cardboard box.

### Compatibility

The MRI Needle Guide SNG is designed for use in combination with the Soteria Remote Controlled Manipulator RCM and for use with biopsy needles with compatible diameters and lengths as listed in Tab.1.

### Application

#### Preparation of the patient

The procedure is to be performed applying aseptic techniques. Further instructions have to be followed according to instructions for use of Soteria Remote Controlled Manipulator RCM.

#### Application of the medical device

1. Check sterile packaging and the product for expiration date and visual damages before use.
2. Open the peel pouch at the end with the markings (see Fig. 1):
  - First open the seal at the two corners from the centre to the outside 
  - Then pull the peel pouch open smoothly  and remove the product aseptically.
3. Use a 10mL disposable syringe with a 21G disposable cannula and fill the MRI Needle Guide SNG with approx. 6mL of sterile water at the proximal filling opening before treatment (see Figs. 3 and 4). The distal end must be completely filled with water and must not contain any visible air bubbles.
4. In order to keep the treatment as comfortable as possible for the patient, it is recommended to use the MRI Needle Guide SNG in combination with a gel (e.g. Lidocaine Gel). After an initial rectal examination, the MRI Needle Guide SNG must be inserted rectally.
5. Connect the MRI Needle Guide SNG to the Soteria Remote Controlled Manipulator RCM in accordance with the instructions for use for the RCM.
6. The MRI Needle Guide SNG is precisely positioned under the MRI imaging according to the Soteria Remote Controlled Manipulator RCM instructions for use.
7. The compatible needle is then guided through the MRI Needle Guide SNG and the biopsy is performed (see Fig. 5).
8. After the procedure, remove MRI Needle Guide SNG from the patient and dispose of in accordance with local provisions and safety regulations.

### Warnings

- The MRI Needle Guide SNG must only be used in combination with Soteria Remote Controlled Manipulator RCM. A combination with other medical products is not approved and can lead to serious injuries to the patient.
- The MRI Needle Guide SNG must not be used and must be disposed of if the product is damaged or if the expiration date has elapsed or if the sterile packaging is damaged or unintentionally opened before use.
- Do not apply extreme force to the tube during application. This may cause the MRI Needle Guide SNG to detach from RCM holder or break.
- The MRI Needle Guide SNG is designed for single use and must not be reused. Reuse and re-sterilization impair safe functionality and can lead to cross-infection of patient or material failure of the product.

## MR safety information

MR Safe

The MRI Needle Guide SNG is not made of any metallic components and thus can be used safely in any MR environment of standard 1.5T and 3T scanners.

### Precautions

- The warranty applies exclusively to the proper intended use of the product.
- The manufacturer does not recommend any specific biopsy needle size and length. Healthcare professional users are responsible for selection of proper size and length depending on patient to be diagnosed.
- Please also note the instructions for use of compatible biopsy needles and the Soteria Remote Controlled Manipulator RCM. The manufacturer assumes no liability and warranty for use with non-compatible medical devices.
- Before use inspect each product for damages at guide tube or connector that would prevent proper functionality. Do not use damaged products.
- According to the European Medical Device Regulation (2017/745) the manufacturer is obligated to inform you of the following:  
All serious incidents that have occurred in relation to the medical device must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is established.  
Note: Serious incidents are understood to mean the death of a patient, user or other person, or the temporary or permanent serious deterioration in the state of health of a patient, user or other person. It does not matter whether these occurred or could occur. The exact definition can be found in MDR 2017/745 Article 2 (65).

### Environmental storage conditions

Temperature		10 – 25 °C
Humidity		20 – 50 % r.h. (non-condensing)
Shelf-life		3 years

#### NOTE:

- Sterility and device functionality are not affected if storage temperature is outside specified environmental conditions over limited short period as long as average storage conditions comply with specifications over respective timeframe.
- For operation no limited environmental conditions apply.
- For transportation packaging can be exposed over limited short period to environmental conditions outside of those as specified for storage.

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©2024 BIP GmbH, Germany, all rights reserved.